EN

Cavifix™ is a product designed to replace a lost filling to assist in case of an emergency. Cavifix™ sets only if it comes into contact with water or saliva and takes approximately 15-20 minutes to set. It has been developed to temporarily seal newly formed cavities and repair lost fillings inlays. It is not a permanent filling material. The composition has been clinically tested. To ensure best results, please read the instruction carefully.

Instruction for use:

1. Wash and dry your hands thoroughly.
2. Put on the glove provided has this will allow a hygienic application of the product.
3. Clean the cavity area with the microbrush provided. **See figure A.**
4. Carefully rinse out the cavity area with warm water.
5. Squeeze out a small amount of the Cavifix™ by using the dispensing key provided. **See figure B.**
6. Roll the product between your fingers and make a small cone. **See figure C.**
7. Press the "cone" firmly into the cavity area. **See figure D.**

8. Remove any excess Cavifix™ product with the point of the spatula provided. **See figure E.**
9. To ensure that you have removed all of the excess Cavifix™ from the cavity area, bite your teeth together gently. This ensures that the Cavifix™ is in even line with the rest of your teeth.
10. The settings of the Cavifix™ take approximately 15-20 minutes in contact with oral moisture. Do not chew for the next 3 hours.

Consult your dentist as soon as possible.

Important:

- You should consult your dentist within 24 hours of applying this product.
- Cavifix™ is only used as a temporary solution.
- Cavifix™ is not to be used on implants, dentures, crowns or bridges.

Attention:

- Cavifix™ should not be used if there is swelling, redness or pain in the affected area. In this case, contact your dentist immediately.

- Cavifix™ should not be applied to open wounds.
- Keep out of reach of children.

Composition:

Calcium Sulphate, Barium Sulphate, Glicol Ether, Zinc Oxide, Acrylic Polymer, Silica.

The kit contains:

1 tube Cavifix™ (7g), 1 dispensing key, 1 plastic-rack, 1 spatula, 1 microbrush, 1 glove, direction of use.

The complete range of BonyPlus products:

Emergency:

Denture "Repairfix": for Denture Repair
"Fixobridge": Fixes Bridges and crowns

Comfort:

"Stabilizer": Soft Denture Relining Cushion
"Superior": 12 Hour Fixative Denture Cream

Hygiene:

"Express" efferescent Cleanser tablets
"Junior" tablets for removable orthodontic appliances
"Denture brush"

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