(EN)

FixoBridge™-Cement is a ready to use cement which fixes dental bridges, temporary crowns or bridges, and all metal, porcelain and acrylic crowns. FixoBridge™ is not a glue - it is a dental-cement which sets only if it comes into contact with water or saliva and can be easily removed by your dentist without any damage to the bridge or crown. Please note that FixoBridge™ is only a temporary solution and you must consult your dentist as soon as possible. The composition has been clinically tested. To ensure, please read the instructions carefully.

Instruction for use:

- Using a strong pin or similar object remove as much as possible of the old, loose cement from inside the bridge, crown or cap. Do not use undue force that may damage the crown. See A
- Using the microbrush supplied, clean the inside of the crown with warm water and soap and rinse it under running water. The crown should preferably be left moist but not dripping wet. See B.
- Using the microbrush clean the area in the mouth (stump) where the bridge, crown or cap is to be replaced. Before replacing the bridge, crown or cap on the stump, rinse the mouth with water (do not fry the stump). See C.

- Apply a small amount of FixoBridge™-3 cement around the inner edge of the crown or cap using the spatula provided. Do not fill the crown. For bridges, apply a small amount of FixoBridge™-3 cement around the inner edge of each crown of the bridge with the spatula provided. See D.
- Press the crown or cap in place for three minutes. For bridges fit the bridge back in your mouth and firmly press down with your thumb and fingers and then carefully bite into place. You will know when the crown or bridge is correctly resisted as you can close your teeth back together again comfortably. See E. (If you are not satisfied with the fit, you have 5 minutes to remove the bridge or crown and repeat the whole process).
- Remove any excess FixoBridge™-3 cement with the microbrush. See F. The setting of the cement takes approximately 15-20 minutes in contact with oral moisture. Do not chew for the next 3 hours.

Important:

- Please remember that FixoBridge™-3 cement takes approximately 15-20 minutes to set in the presence of oral moisture - it is not a glue.

- Always use small quantities and apply to the inner edge of the crown.

Attention:

- Store out of reach of children.
- Store in a cool, dry place (5-25°C / 41-77°F).
- It is important that the cap is replaced tightly after each use.

The kit contains:

- 1 tube of Cement, 1 spatula, 1 plastic-rack, 3 microbrushes, directions for use.

The complete range of BonyPlus products:

- Emergency, Denture Réparatif, Cavité, Confort, Denture Relining Cushion - Stabilizer, 12 Hour Special Fixative Denture Cream
- Hygiene:** "Express" effervescent denture cleanser tablets, "OrthoJunior" tablets for removable orthodontic appliances, Denture brush.

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