## Effervescent tablets for disinfecting removable dental appliances

### EN Oral care for diabetic patients

Dental appliances are porous and, upon first-use, can become colonized by pathogenic micro-organisms. A biofilm forms on the appliance over time and this can affect oral hygiene and it becomes important to maintain and disinfect your removable dental appliance. Suffering from a compromised immune system makes it even more important to maintain a good oral hygiene.

### What is a biofilm ?

Biofilms can be defined as communities of micro-organisms attached to a surface. Due to their porous nature and to the ideal conditions provided by the oral cavity, removable dental appliances are especially susceptible to being contaminated by a biofilm. There are several species of micro-organisms that may comprise a biofilm, notably, Candida albicans, Staphylococcus aureus, Pseudomonas aeruginosa, Pseudomonas fluorescens, Escherichia coli, Vibrio cholerae, and the Herpes simplex virus.

### Compromised immune system ?

The DiabetiXPro tablet is proven scientifically to not only kill but also remove potentially harmful micro-organisms including Candida albicans from deep within the micro-porosities of the appliance.

Ensuring that your removable dental appliance is kept clean and free of biofilms will avoid any additional strain on an already compromised immune system.

**Eradicates** potentially harmful biofilm on your removable dental appliance.

**Prevents** reinfection through infected appliances.

Clean and fresh feeling & restoration good oral hygiene.

#### ISO 13727 - Bactericidal Activity in 15 minutes

Pseudomonas aeruginosa, Staphylococcus aureus and Enterococcus hirae

#### ISO 13624 - Yeastcidal activity in 15 minutes

Reduces Candida albicans

#### ISO 14476 + A1 Virucidal activity

Significantly reduces Adenovirus and Poliovirus in 1 hour  
A single use of NitrAdine® (DiabetiXPro) is effective in eliminating certain species of micro-organisms, including selected viruses (Herpes Simplex), in vitro. (Quintessence Int 2004, Prof. R. Thomas Glass, et al.) A novel disinfecting formula for the maintenance of oral medical devices... (Journal of Applied Microbiology, Prof. T. Coenye)

### Possible symptoms due to a contaminated appliance

- Burning sensation in the mouth
- General mouth soreness and discomfort
- Changes in tastes and bad breath (halitosis)
- Cuts on the corners of your mouth and lips (Angular cheilitis)

### RELIEVE DISCOMFORT INDUCED BY AN INFECTED REMOVABLE DENTAL APPLIANCE

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# DiabetiXPro®

**EN • Precautions for use:** For external use only. Discard solution after use. Do not use if allergic to any of the ingredients or to persulfates. Do not put tablet in mouth. Tablet powder may cause eye damage. Do not drink the solution or use as a mouthwash. Rinse denture well before replacing it in your mouth. Discontinue use and seek medical attention if any irritation, swelling or pain develops following the first or any later use. Consult instructions for your use. Store in a dry place. For single use only.

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**EN** Before using the DiabetiXPro tablet, brush, cleanse and rinse the removable dental appliance as usual.

Immerse the appliance in a glass of lukewarm water (approx. 150ml).

Add one tablet. Use only one tablet per appliance.

Make sure the appliance is fully covered with water. For regular bacteria and yeast infection, allow the appliance to soak for 15 minutes in the solution. To ensure that your appliance is free of all viruses, soak it for 1 hour in the solution. You may soak the appliance overnight without damaging the appliance.

Remove the appliance from the solution and rinse well under water before replacing it in the mouth. If a taste should remain on your appliance after replacing it in your mouth, place it in lukewarm water for 10 minutes.

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## Frequency of use

**Recommended dosage:**  
One tablet two times per week.

### Intensive treatment:

If recommended by your dentist/orthodontist, use the tablet initially every day for two weeks. After this two week period has elapsed, use two times per week as part of your cleansing routine.

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**EN • Dry tablet:** Causes skin irritation. Causes serious eye damage. Keep out of reach of children. Wash hands thoroughly after handling. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. **IF ON SKIN:** Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

Contains Pentapotassium bis(peroxymonosulphate) bis(sulphate). Contains Dipotassium peroxodisulphate. May produce an allergic reaction. Contains Corment oil/Mentha arvensis oil. May produce an allergic reaction.

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**bonyf AG**