www.diabetixpro.com

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### EN Oral care for diabetic patients

Olive Oil's health properties and benefits have been studied for decades. DiabetiXPro denture fixative cream is the first denture cream of its kind to use Olive Oil as a replacement for the commonly used petrochemicals.

Patients suffering from immune depressing conditions that can affect the oral cavities will surely feel the difference not only in terms of comfort but also in strength and duration of hold. They will also derive some of the health benefits of the unique Olive Oil based formulation, which contains high concentrations of Oleic and Linoleic acids, known for their antibacterial & fungistatic properties.

### Antibacterial Properties of the Olive Oil

The fixative cream contains up to 35% Olive Oil and due to this contains up to 17% Oleic Acid & 7% Linoleic Acid - known for their fungistatic & antibacterial properties.

### Safe according to clinical-dental & clinical-dermatological criteria

„ From a dermatological and dental point of view it can hence be concluded that the product was tolerated very well in a four-week application with 20 subjects tested according to clinical-dermatological, clinical-dental and subjective criteria. ”

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### Ingredients :

Cellulose Gum, Olea Europea (Olive Fruit) Oil, Calcium/Sodium PVM/MA Copolymer, Hydrogenated Soybean Oil, Trihydroxystearin, Silica, Menthol, Lecithin, Citrus Limon Peel Oil, Menthyl Lactate.

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# DiabetiX Pro®

1 Rinse and clean your dentures as usual.

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2 Remove the hygienic seal from the tube nozzle.

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3 Humidify your appliance and apply the cream as shown.

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4 Before placing the appliance back in the mouth drink some water to humidify the gums.

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5 Place your appliance in the mouth and maintain it in place by pressing down for a few seconds.

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6 Wait 5 mins. before drinking or eating.

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## Important Information for Optimal Use

Due to the relatively high concentration of olive oil in the formula (up to 35%) you may notice some drops of oil that separate from the cream. This is normal and should in no way affect the performance of the cream. Please follow the instructions below to avoid this problem:

It is recommended to knead the tube before use (between 10 and 30 seconds).

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## EN Precautions for use and storage

After use, you may notice some cream residue on your appliance. This can be easily removed with a cotton bud soaked in lukewarm olive oil.

You can adapt the quantity of cream without any risk. Lower dentures usually require more cream for optimal holding strength and duration.

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**Storage:** Avoid exposing the tube to direct sunlight or other sources of heat.

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**Caution:** Do not use product if tube nozzle is unsealed. Use the product according to instructions for use. Keep out of the reach of children. Do not use if there is a known sensitivity or allergy to one of the ingredients. If you are taking medication that may affect your gums or oral cavity, consult a dentist before using the product. In the event of gum irritation or soreness, discontinue use and consult your dentist if necessary. Pain and discomfort of the gums can also be caused by ill-fitting or broken dentures therefore it is important to consult your dentist on a regular basis. Store dry and at room temperature.

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