

SEE INSTRUCTIONS ON NEXT PAGE

BONYPLUS® SMILE WITH CONFIDENCE™ *Denture Relining Cushion Stabilizer*

- Eliminates pain, wobbling, pressure points
- Only apply once a month, no more daily denture adhesives
- Vacuum grip

With SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* your denture fits like a glove. The application of SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* may seem a little difficult at first but the reward for your effort is that you will have no problems with your denture for 1 month. Probably no more daily denture adhesives, no more pain, no more wobbling and no more pressure points. If you have problems with application due to health or age, please ask a trusted person for help.

PLEASE NOTE:

- Use only for complete plastic dentures – not for metal dentures
- 1 pack of SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* is only for 1 denture. If you have problems with upper and lower dentures you need 1 pack of SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* for each denture.

THE ADVANTAGES OF

SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*

- Creates an exact copy of upper and lower gum.
- Keeps its soft consistency for 4 weeks (no pressure, no pain).
- Only needs to be reapplied once every 4 weeks.
- SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* is flavorless and odorless.
- Doesn't come off the denture while eating or drinking.
- The denture can be taken out any time for cleaning.
- Helps prevent your denture from breaking.

PLEASE NOTE AFTER APPLICATION

- Do not eat food for 1 hour after applying SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* (drinking is allowed).
- Clean your denture with a soft denture brush so you do not damage the SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*.
- Consult your dentist if SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* does not bring any improvement.
- Use the intensive effervescent cleansing tablets from SMILE WITH CONFIDENCE. They have been specially developed to keep the softness and consistency of SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*.

CONTENTS

- 1 phial BONYPLUS 45 liquid 2.5 ml: (Triacetin, Acetyl Tributyl Citrate, Alcohol)
- 1 phial BONYPLUS 45 powder 2.95 g: (Polyethylacrylate, CI 20055, CI 20060, Titanium Dioxide)
- 1 phial BONYPLUS 45 separator liquid 0.5 ml: (Shellac, Alcohol, Acetone)
- 1 mixing dish
- 1 spatula
- 1 brush
- 1 direction leaflet
- 1 transparent plastic tray

REMOVE SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* (without fail after 4 weeks)

- Leave SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* on your denture for no longer than 1 month otherwise it will become hard and difficult to remove. For this reason it is essential that you renew SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*, every 4 weeks.
- **For removal:** Put your denture in 122° F (50° C) water for 10 minutes. (Do not put in water above 122° F 50° C)
- To remove SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* use the help of a blunt knife.

RE M I N D E R :

SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* should be removed from your denture after 4 weeks before it becomes hard and difficult to remove. Make a note of the date on which you should remove the SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*

APPLIED SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*

Date: _____

REMOVE SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer*

Date: _____

PLEASE NOTE:

- Keep the box dry and at a temperature below 77° F (25° C)
- Keep out of reach of children
- Do not use if you have a known allergy to any of the ingredients
- Do not bring SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* in contact with open wounds
- Rare allergic reactions to ingredients are possible
- Flammable

OTHER SMILE WITH CONFIDENCE CONVENIENCE PRODUCTS:

EMERGENCY

SOS Denture Repair-fix and Monodent Denture Repair-kit

Instant repair kits for broken or cracked dentures and detached teeth.

Fixobridge

Replaces dislodged crowns and bridges

Replaces temporary resin bridges

Cavifix – Temporary Dental Filling Material

Multiple use:

Seals temporarily cavities from lost fillings

Seals temporarily newly formed cavities

Reduces discomfort from sharp cavity edges

Restores visual appeal

COMFORT

12 Hour Special Fixative Denture Cream

Special Formula:

Cream activates in the mouth for an exceptionally Strong

Lasting Hold

Denture Relining Cushion

Relining kit improves fit and comfort of full dentures

Relieves painful pressure spots and sore gums

1 application lasts 4 weeks

HYGIENE

Express Effervescent Denture Cleanser Tablets in 32's

Faster than regular tablets – Easy to use – Highly effervescent cleansing power – Removes stains and residues – Restores artificial teeth colour – Reduces plaque build up – Whitens & brightens dentures like new, guaranteed – Prevents denture odour – Leaves a fresh minty flavor – Removes food particles that cause irritation.

Denture Toothpaste

Polishes and cleans – Refreshes – Bacteria & Stain Reduction

Denture Brush

Also Available: BONYPLUS/SMILE WITH CONFIDENCE

Junior Tablets in 40's

Specific effervescent tablets for removable orthodontic appliances

Steryl Brush Tabs in 16's

Effervescent tablets for a hygienic toothbrush

Medical in 20's

Disinfecting denture tablet for treatment of denture thrush

Product of:
bonyf AG © CE
Heiligkreuz 16
FL-9490 Vaduz
Liechtenstein – Europe
www.bonyf.com

6.700.01.D.0.0.01

MADE IN SWITZERLAND

Dental Line



BONYPLUS®
THE DENTURE & DENTAL SPECIALIST

SMILE WITH CONFIDENCE™ *Denture Relining Cushion Stabilizer* INSTRUCTIONS: READ CAREFULLY

1

NOTE: If you are a regular user of SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* check first that there is no prior glossy layer of "SEPARATOR" visible on your denture. **If there is DO NOT APPLY** another layer of "SEPARATOR"; use it the next time. The "SEPARATOR" is used only to enable the SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* mixture to be removed easily after 4 weeks.

- Picture 1.1** Clean and dry your denture.
Picture 1.2 Open the small bottle labelled "SEPARATOR" and stand it upright in the transparent plastic tray.



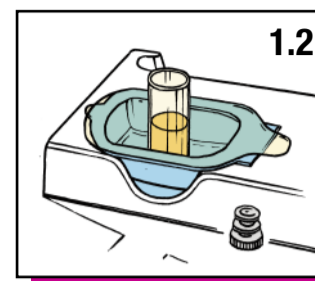
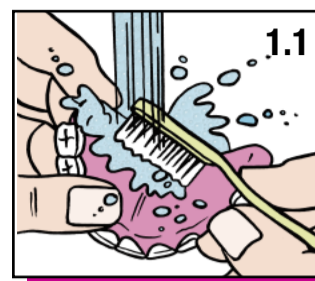


2

ATTENTION: Only apply thinly: you can check this by the glossy transparent appearance of the "SEPARATOR" liquid. Use only a very small amount of the liquid in the bottle. Allow to dry for 10 minutes.


Please paint the "SEPARATOR" fluid thinly and evenly (use paintbrush) over the inside of your denture.

- Picture 2.1** Upper denture: keep 5 mm (approximately 1/5 inch) from the back and front inside edges (see shaded area on picture).
Picture 2.2 Lower denture: also keep 5 mm (approximately 1/5 inch) from the front and inside edges.
Picture 2.3 **IMPORTANT:** If your lower gums are very flat and your denture has a small or no groove (see picture 2.3) do not apply the "SEPARATOR".
Picture 2.4 Lower denture with normal or deep groove: apply "SEPARATOR", also stay away 5 mm from the front and inside edges.

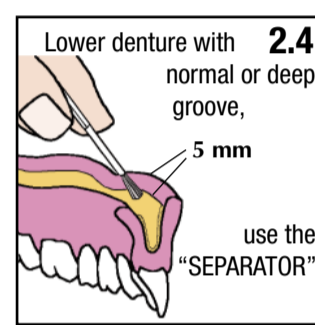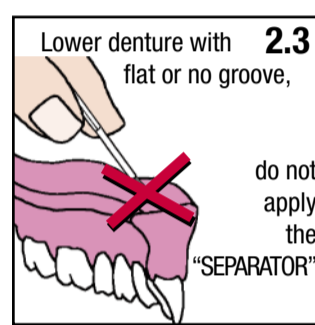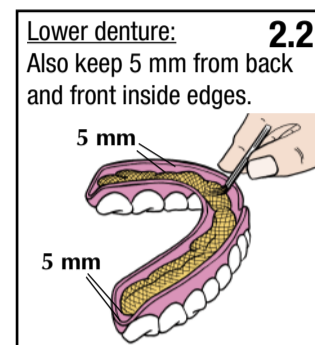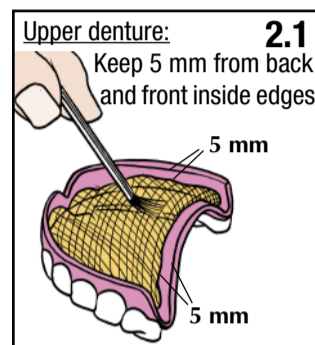

3

- Picture 3.1** Set the transparent plastic tray, brush and separator bottle aside. These can now be thrown away.


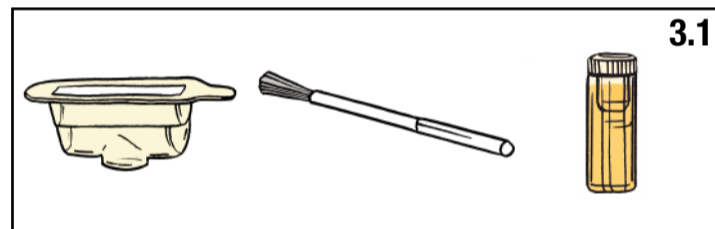


4

While mixing keep the blue mixing dish in the white plastic tray to maintain a better hold. Only take out to pour.

IMPORTANT: DO NOT mix partial amounts, use all of the powder and liquid, otherwise you will not get a fluid mass and it will not be possible to use SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* any more.



- Picture 4.1** Pour first the whole quantity of powder, followed by the whole quantity of liquid into the blue mixing dish and with the Spatula provided mix until a thick fluid mass is produced.
Picture 4.2 Immediately go to step 5.

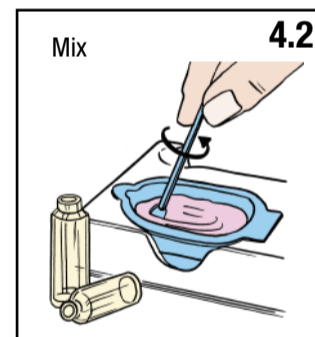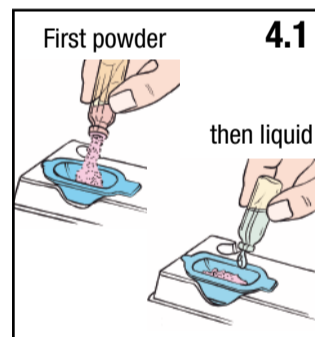

5

- Picture 5.1** Now pour the thick SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* fluid mixture immediately into the groove of your denture.



TIP For the upper denture: Pour the SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* mixture initially into the middle of the groove (see picture 5.1) for ease of uniform spreading. Use the blue spatula to assist in removing the mixture from the mixing dish and to spread it over the denture.

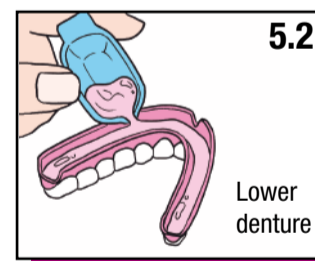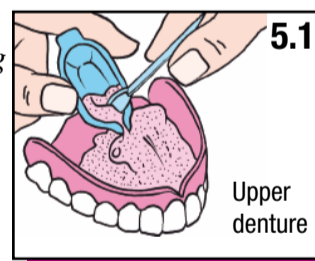

6

- Picture 6.1** Now put the denture in your mouth. Tilt your head slightly forward to avoid the mixture running off the back of the denture into your throat.



- Picture 6.2** Now press your teeth together gently and hold for about 4 minutes. Breathe through your nose. During this time move your cheek muscles and lips lightly with your mouth closed.

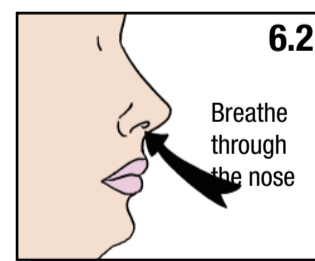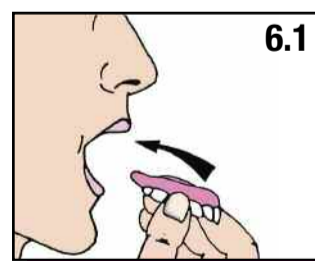

7

Now gently remove the denture from your mouth.

- Picture 7.1** Upper or lower denture: scrape off excess mixture overlapping the teeth with your thumbnail.
Picture 7.2 Upper denture: cut excess mixture off the end of the denture downwards with a sharp knife.

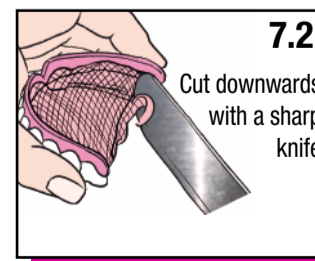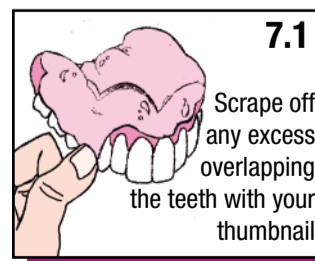

NOW THE APPLICATION IS COMPLETE: THE SMILE WITH CONFIDENCE *Denture Relining Cushion Stabilizer* DOES NOT HAVE TO BE APPLIED AGAIN FOR 4 WEEKS.