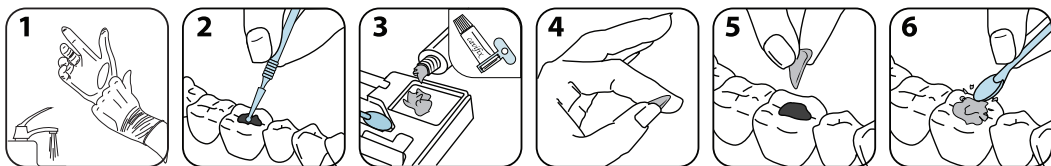
BONYPLUS® Cavifix™

DE

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Manufactured by:

**Advanced
Healthcare Ltd**
TN11 8JU
United Kingdom

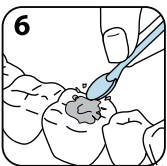
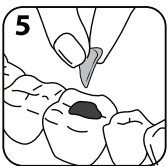
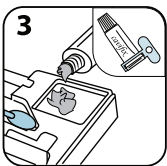
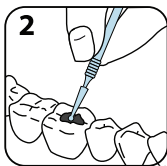
Supplier for distribution:

bonyf AG
Heiligkreuz 16
FL-9490 Vaduz
Liechtenstein - EUROPE
www.bonyf.com

Ingredients: Calcium Sulphate, Barium Sulphate, Glycol Ether, Zinc Oxide, Acrylic Polymer, Silica.

SWISS CONTROLLED QUALITY

**EN**

Cavifix™ is a clinically tested ready to use synthetic resin based filling material designed to temporarily seal newly formed cavities and repair lost fillings or inlays. Cavifix™ is not a permanent filling material and is ideal for use in emergency cases to replace or repair a lost filling. It is of adequate strength, yet can be easily removed. The product is safe and will not irritate your gums. Cavifix™ contains the same ingredients dentists use.

Recommended usage: 1 application lasts for up to 4 weeks.

Caution: Read the enclosed directions carefully before starting the repair. Cavifix™ should not be used if there is swelling, redness or pain in the affected area. In that case contact your dentist immediately. Do not apply to open wounds. Keep out of reach of children. Do not ingest the product. Store in a cool dry place (5–25°C). Replace the cap tightly after each use. Do not use if there is a known sensitivity or allergy to one of the ingredients.

Package contents: 1 tube Cavifix™ 7g, 1 dispensing key, 1 spatula, 1 microbrush, 1 glove, 1 directions for use.

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Instructions for use:

Cavifix™ sets only if it comes into contact with water or saliva and takes approximately 30–45 minutes to set.

To ensure best results, please read the instructions carefully.

1. Wash and dry your hands thoroughly. Place the glove on your hand to allow for a hygienic application of the product. **See diagram 1.**
2. Clean the cavity area with the microbrush provided and rinse out the area with warm water. **See diagram 2.**
3. Squeeze out a small amount of the Cavifix™ on the flat space on the plastic inlay holder by using the dispensing key provided. **See diagram 3.**
4. Roll the product between your fingers to form a small cone shape. **See diagram 4.**
5. Press the "cone" firmly into the cavity area. **See diagram 5.**
6. Remove any excess Cavifix™ product with the point of the spatula provided. **See diagram 6.**

Please note: To ensure that you have removed all of the excess Cavifix™ from the cavity area, bite your teeth together gently. This ensures that the Cavifix™ is in even line with the rest of your teeth. The setting of the Cavifix™ takes approximately 30–45 minutes in contact with oral moisture. Do not chew for 1 hour. Consult your dentist as soon as possible.

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