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Please read all pages of the following instructions for use thoroughly BEFORE carrying out the repair. Be patient and take your time to ensure the repair is successfully carried out. **IMPORTANT: FOR EMERGENCY USE ONLY. THIS IS A TEMPORARY REPAIR. CONSULT YOUR DENTIST AT THE EARLIEST OPPORTUNITY FOR A MORE PERMANENT SOLUTION.**

EN Read before using

This product is an emergency denture repair system for removable synthetic resin dentures. It is for temporary use only. Visit your dentist for a more permanent repair. Do not use the product for repairing dental crowns, jacket crowns, bridges, fixed dentures or dentures for cleft palate patients. If the repair is not successful, consult your dentist as soon as possible.

If you feel severe pain, irritation or swelling following the use of the repaired denture, discontinue use and consult your dentist/doctor. Keep out of the reach of children. Do not use the product if you are allergic to any of the components listed on the instruction sheet or the product box.

Make sure to carry out the repair on the side of the denture that is not pressing against the palate/gums.

Avoid bringing the product in contact with gaping wounds. It is recommended to clean your denture on a daily basis. This product is not recommended for use on children.

Storage

Store in a cool and dark place (under 25°C).

INSTRUCTIONS FOR USE

Make sure to carry out the repair on the side of the denture that is not pressing against the palate/gums.

1. Before application of Denture Repair-fix, clean and brush your broken denture pieces or place them in a glass of lukewarm water with a denture cleanser tablet. Rinse your denture after three minutes and dry it carefully.

2. Put the two broken denture parts into their original form. In case the denture form is incomplete, do not carry out the repair and consult your dentist immediately.

3. Take the grey tube marked BONYPLUS® 1a, unscrew the cap (A) and unscrew the nozzle (B). Use the sharp point on top of the cap (A) to perforate the seal on the tube. Avoid squeezing the tube once it has been perforated. Screw the nozzle (B) and cap (A) back onto the tube.

4. Unscrew the cap and apply a small quantity of BONYPLUS® 1a glue along the whole length of the cracked edge on one of the broken denture pieces.

5. Gently press both parts of the denture together to restore the original denture form and hold for 1 minute.

6. Any surplus glue remaining on your denture can be removed by wiping it with a cloth on both sides of the denture.

7. Ensure the glue has cured by gently probing the repaired part of the denture. Place the denture in your mouth to evaluate the accuracy of the repaired denture form. If you are dissatisfied with the fit, disassemble the repaired pieces and repeat steps 2 to 6. Take particular care with Point 5.

8. Remove the denture from your mouth, rinse and dry it carefully.

9. Slightly rough up the surface of the denture (tongue side) by gently rubbing it along the crack (approx. 1cm either side of the crack) using the emery paper.

10. Take the vial marked BONYPLUS® 3, shake it well and pour the pink powder into the mixing cup.

11. Take the vial marked BONYPLUS® 2 and the spatula. Pour the liquid slowly into the powder by slightly tapping the vial and gently stir the mixture together using the spatula. **IMPORTANT:** Leave a small quantity of the liquid remaining in the vial for the final smoothing (see Figure 15).

12. Mix the components for approx. 15 seconds until the paste is smooth and homogeneous.

13. Let the paste stand for approx. 1 minute in winter and a maximum of 15 seconds in summer. The paste must remain smooth in order to facilitate an even application.

14. Use the spatula to apply the paste on the entire length of the previously roughened area of the denture. The paste layer should have a minimum thickness of 2mm (see Figure 15a). Ensure that the application of the paste does not take more than one minute in order to facilitate the smoothing process (see next step).

15. Put the latex finger glove on your index finger and moisten it with a small quantity of the remaining BONYPLUS® 2 liquid contained in the vial. Use your finger to smooth over the previously applied paste.

16. IMPORTANT: READ THE FOLLOWING SENTENCES CAREFULLY.

Allow the denture to set for at least 15 minutes before inserting in your mouth to ensure that the paste mixture has hardened, and is not still in liquid form and soft. Use your finger to check that the paste you have applied on your denture in Point 14 is hard. If the paste is not completely hard and the denture is placed in your mouth, the paste can stick to your teeth and/or your gums.

Your denture is now ready to use and you can once again eat and smile with confidence. **Please note this is a temporary repair only and you will need to consult your dentist for a more permanent solution.**

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