
# BONYPLUS® Fixobridge™

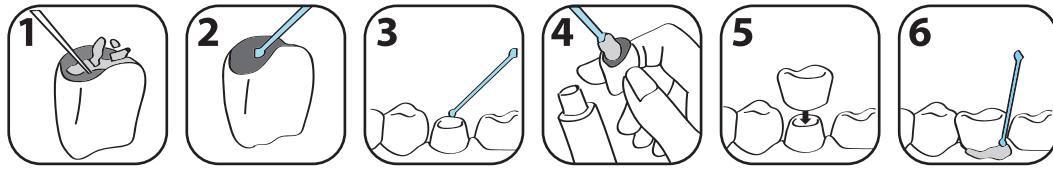## DE

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Manufactured by:

Advanced Healthcare Ltd  
TN11 8JU  
United Kingdom




4.700.00.A.0.0.99 - R05042018D-PT

Supplier for distribution:

bonyf AG  
Heiligkreuz 16  
FL-9490 Vaduz  
Liechtenstein - EUROPE  
[www.bonyf.com](http://www.bonyf.com)

Ingredients: Calcium Sulphate, Glycol Ether, Barium Sulphate, Zinc Oxide, Acrylic Polymer, Triacetin.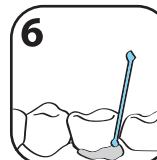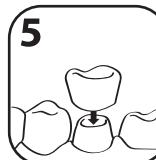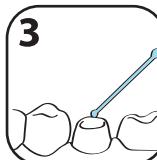

EN

**Fixobridge™-3cement** is a ready to use cement which re-fixes dental bridges, temporary crowns or bridges, and all metal, porcelain and acrylic crowns. **Fixobridge™** is not a glue - it is a dental-cement which sets only if it comes into contact with water or saliva and takes approximately 15-20 minutes to set. **Fixobridge™-3cement** has been specially developed to provide an ideal temporary solution. It does not irritate the gums, it is easy to work with and can be easily removed by your dentist without any damage to the bridge or crown. Please note that **Fixobridge™** is only a temporary solution and you must consult your dentist as soon as possible. The composition has been clinically tested. To ensure correct application of the product, please read the instructions carefully.

**Caution:** Read the enclosed directions carefully before using. Keep out of reach of children. **Fixobridge™** is a temporary solution only and you must consult your dentist as soon as possible. **Fixobridge™** should not be applied to open wounds. Do not ingest the product. Store in a cool, dry place (5-25°C). It is important that the cap is replaced tightly after each use. Do not use if there is a known sensitivity or allergy to one of the ingredients.

**Package contents:** 1 Tube of **Fixobridge™-3cement** 7g, 1 Spatula, 3 Microbrushes, 1 Instructions for use.

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#### Instruction for use:

- Using a strong pin or similar object remove as much as possible of the old, loose cement from inside the bridge, crown or cap. Do not use undue force that may damage the crown. See diagram 1.
- Using the microbrush provided, clean the inside of the crown with warm water and soap and rinse it under running water. It is preferable to leave the crown moist, but not dripping wet. See diagram 2.
- Using a new microbrush clean the area in the mouth (tooth stump) where the bridge, crown or cap is to be replaced. Before replacing the bridge, crown or cap on the stump, rinse the mouth with water (do not dry the stump). See diagram 3.
- Apply a small amount of **Fixobridge™-3cement** around the inner edge of the crown or cap using the spatula provided. Do not fill the crown. For bridges, apply a small amount of **Fixobridge™-3cement** around the inner edge of each crown of the bridge with the spatula provided. See diagram 4.
- Press the crown or cap in place for three (3) minutes. For bridges: fit the bridge back in your mouth and firmly press down with your thumb and fingers and then carefully bite into place. You will know when the crown or bridge is correctly re-seated as you should be able to close your teeth back together again comfortably. See diagram 5. (If you are not satisfied with the fit, you have 5 minutes to remove the bridge or crown and repeat the whole process).
- Remove any excess **Fixobridge™-3cement** with the remaining microbrush. See diagram 6. The setting of the cement takes approximately 15-20 minutes in contact with oral moisture. Do not chew for the next 3 hours.

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