

This product can be used on the following appliances: removable dentures, removable orthodontic appliances, anti-bruxism and anti-snoring devices (mandibular advancement devices).

## Packaging

Product available in the following tablet box sizes: 12, 20, 32 and 64. The number of tablets contained in this package can be found on the front of the product box.

## Precautions for use

For external use only. Discard solution after use. Do not use if allergic to any of the ingredients or to persulfates. Do not put tablet in mouth. Tablet powder may cause eye damage. Do not drink the solution or use as a mouthwash. Rinse appliance well before replacing it in your mouth. Discontinue use and seek medical attention if any irritation, swelling or pain develops following the first or any later use. Consult instructions for use. Store in a dry place. For single use only.



Danger

*Causes skin irritation. Causes serious eye damage. Keep out of reach of children. Wash hands thoroughly after handling. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor. IF ON SKIN: Wash with plenty of water. If skin irritation occurs: Get medical advice/attention. Contains Sodium lauryl sulfate; Pentapotassium bis(peroxymonosulphate) bis(sulphate). Contains Potassium persulfate and 2-Isopropyl-5-methylcyclohexanone. May produce an allergic reaction.*

## Instructions for use

1. Rinse your appliance under running water. If necessary use a toothbrush or denture brush to remove any food debris.
2. Place the appliance in a glass of 150 ml of warm water (33°C - 47°C) ensuring that it is fully immersed and immediately place 1 tablet in the water. Soak for a minimum of 15 mins to ensure reduction of the biofilm. Acrylic appliances can be soaked overnight.

**NB: If your tablet is broken the efficacy and usage of the product will not be affected (provided the whole tablet is used).**

3. Remove the appliance from the solution and rinse well under water before replacing it in the mouth. If a taste should remain on your appliance after replacing it in your mouth, place it in warm water for 10 mins.

**Recommended dosage:** Based on clinical testing, we recommend to begin the treatment with a daily use of the tablet for 21 days. This will ensure the reduction of the biofilm to a lower level. You can then adjust the usage based on your needs with a minimum usage of 2 times per week. This product has a proven efficacy against *Candida albicans*, *P. aeruginosa*, and *S. aureus*.

**Conditions of the Disinfecting activity testing:** The bactericidal and yeasticidal tests were performed using 1 tablet in 150 ml of lukewarm hard water (at 37°C) soaking for 15 minutes under clean conditions in 80% final concentration / low load level. The virucidal test was performed using 1 tablet in 150ml of hard water (at 20°C) soaking for 15 minutes under clean conditions in 97% final concentration / low load level.

**Ingredients:** Citric Acid, Sodium Lauryl Sulfate, Lactose Monohydrate, Sodium Bicarbonate, Sodium Chloride, Potassium Monopersulfate, Sodium Carbonate, Peppermint Flavour, PVP.

Manufactured by:



Heiligkreuz 16  
FL-9490 Vaduz  
Liechtenstein - EUROPE  
Tel: +423 232 78 15  
www.bonyf.com



Swiss Made



DO NOT  
RE-USE



DO NOT  
DRINK



DO NOT  
INGEST



CAUTION



CONSULT  
INSTRUCTIONS  
FOR USE



KEEP OUT  
OF REACH OF  
CHILDREN



RINSE  
BEFORE  
USE



KEEP  
DRY