FR

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EN

Revolutionary & Healthy Approach No petrochemicals

The application of a denture adhesive cream implies its absorption through the mouth, into the body. This is why we have decided to replace oils derived from petrochemicals (petrolatum, mineral oil and vaseline), commonly used in the formulation of denture adhesive creams on the market, by a healthier alternative, Olive oil.

- Long lasting hold 24h*
- Silky smooth hold

A test was conducted with denture wearers by Dermatest (Germany) to assess the effectiveness of OlivaFix. The results were a success and showed that not only the cream was dermatologically safe but also had a strong hold, depending on the type of denture.

Due to the high concentration of Olive oil, OlivaFix has a silky consistency and spreads smoothly on the appliance creating a protective film for the gums.

- 0% Silicones
- 0% Colour additives
- 0% Preservatives
- 0% Zinc
- 0% Parabens
- 0% Petrochemicals

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**TENUE EXTRA FORTE
EXTRA STRONG HOLD**

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EN Useful information

If cream residues remain on your dentures or your gums after removal of the dentures, use a clean paper towel to wipe the cream residue away.

You can safely adapt the quantity of cream used: lower dentures or flat gums usually require more cream.

Storage: Do not expose the tube to sources of heat or direct sunlight.

Due to the high concentration of Olive oil (up to 35%) it is possible that you notice some Olive oil residue during application. This will in no way affect the performance of the cream nor does it pose any risk to your appliance. In order to minimise these residues, please follow the advice below.

It is recommended to knead the tube before use (between 10 and 30 seconds).

Caution: Do not use product if tube nozzle is unsealed. Use the product according to instructions for use. Keep out of the reach of children. Do not use if there is a known sensitivity or allergy to one of the ingredients. If you are taking medication that may affect your gums or oral cavity, consult a dentist before using the product. In the event of gum irritation or soreness, discontinue use and consult your dentist if necessary. Pain and discomfort of the gums can also be caused by ill-fitting or broken dentures therefore it is important to consult your dentist on a regular basis. Store dry and at room temperature.

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