

# BONYPLUS®

## Superior

### Denture adhesive cream

✓ No Zinc ✓ Easy application ✓ Strong hold



### Instructions for use

**Please read caution text before use.**

1. Before applying the cream clean your denture as usual.
2. Remove the metallic anti-humidity & hygienic seal from the tube.
3. Apply the cream on a dry denture as shown in the picture.
4. Gently humidify your denture with water before placing it in your mouth and drink a sip of water to humidify the gums. This is especially recommended for patients who suffer from a dry mouth, as the cream activates with moisture or saliva.
5. Place your dentures in the mouth and hold them in place by maintaining pressure for a few seconds.
6. Wait 5 minutes before drinking or eating.



**UPPER**



**LOWER**

**Note:** If the cream is difficult to squeeze out of the tube, place it in a glass of warm water (up to 50°C) for a short period of time, making sure that the cap is not submerged. If the opening of the tube becomes clogged, use a clean pin to unblock it. Keep the nozzle of the tube dry and free of any excess cream to prevent clogging. If you have any difficulty in removing your denture after the adhesive cream has been applied, rinse your mouth with warm water.

**Caution:** Do not use product if tube nozzle is unsealed. Use the product according to instructions for use. Keep out of the reach of children. Do not use if there is a known sensitivity or allergy to one of the ingredients. If you are taking medication that may affect your gums or oral cavity, consult a dentist before using the product. In the event of gum irritation or soreness, discontinue use and consult your dentist if necessary. Pain and discomfort of the gums can also be caused by ill-fitting or broken dentures therefore it is important to consult your dentist on a regular basis. Store dry and at room temperature. If any adhesive cream residues remain on your gums after you have removed your denture, rinse your mouth with warm water or gently brush your gums with toothpaste.

**Ingredients:** Cellulose Gum, Petrolatum, Paraffinum Liquidum, Calcium/Sodium PVM/MA Copolymer, Silica Dimethyl Silylate, Silica, Menthol, Menthyl Lactate, Lecithin.

Product available in the following sizes: 20g, 40g, 68g, 70g and 75g. The weight of this product can be found on the box.

### Manufactured by:

bonyf AG   
Heiligkreuz 16  
FL-9490 Vaduz  
Liechtenstein  
[www.bonyf.com](http://www.bonyf.com)



KEEP OUT OF REACH  
OF CHILDREN



CONSULT INSTRUCTIONS  
FOR USE



KEEP DRY



CAUTION

pH: 4-10



**MADE IN SWITZERLAND**

5.741.00.A.0.0.01 - R17022023D-PT